



1. Transfer all medication information (including warnings and cautions) to provided blank label using a ball point pen and apply label to Trak-a-Dose™ vial.

Date _____	Expiration _____
Patient _____	
Medication _____	
Directions _____	
Warnings _____	
Doctor _____	
Pharmacy _____	
Rx Number _____	

*Save the original medication container for reference

*DO NOT USE this product without a properly filled out label on vial. Medication listed on label should match medication inside vial.



Failure to accurately transfer information from the original medicine container could result in serious injury or death.

2. When next dose of medication is due, remove marker and erase all prior information on dry erase writing surface.

3. Open vial and take medication as directed.

CHILD RESISTANT

Close Lid then
PUSH DOWN AND TURN
←Close Open→

EASY OPEN


CAUTION NOT CHILD RESISTANT
To Open, close lid pull cap




Failure to accurately follow directions from your medical professional could result in serious injury or death.

4. IMMEDIATELY AFTER TAKING MEDICATION, write the current time (the time you took the medication) and circle "AM" or "PM" under "Last Dose Taken". Write the current date on the dry erase surface.

CHILD RESISTANT

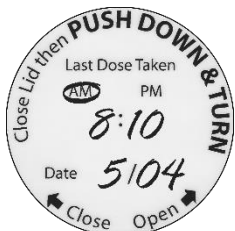

EASY OPEN

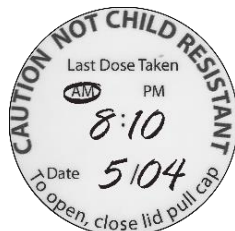

*Examples of medication taken at 8:10 AM on May 4

5. Review the information on the writing surface as needed to help ensure proper dosing. Repeat steps when the next dose is due.

⚠ WARNING

- Take medication as directed
- Keep away from children